SCHEDA TECNICA – 4+ MASK COD. R453

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Double protection: protects you from others and others from you

DESCRIPTION

4+ MASK surgical mask with **double protection**: protects you from others and others from you, thanks to 4 functional layers:

- Additional layer made of a film of hexagonal structure polyethylene net, as second layer of the mask, which makes the mask resistant to splashes of potentially contaminated liquids, which are a possible transmission carrier of infective viruses.
- 2nd coloured outer layer of cellulose and polyester, with oriented fibres
- 3rd filtering layer of Melt blown polypropylene fibrils
- 4th anti-macerating inner layer (in contact with the face) of polypropylene-polyethylene

Type IIR mask with Filtering efficiency BFE. $\geq 99\%$ (norm required $\geq 98\%$).

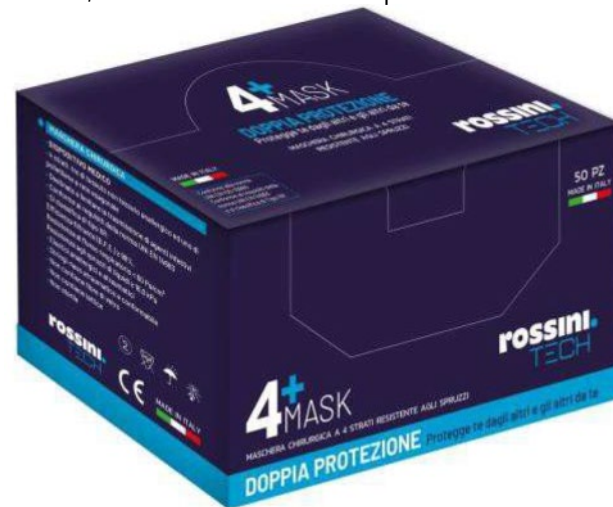
Comfortable tubular ear-loops, made of polyester and elastomer, non-allergic and nuisance-free

Modelable nose clip along the entire length of the mask, which ensures optimal breathing.

PACKAGE

of 50 pieces per carton

900 pieces per box (18 packaging)

rossini.
TECH

MEDICAL DEVICE
EN 14683:2019+AC:2019
EN ISO 10993-5:2009
EN ISO 10993-10:2013

CERTIFICATIONS

4+ MASK is a Type IIR surgical mask, according to the requirements of the norm UNI EN 14683:2019, it is a Class I medical device (as per 93/42/EEC directive), self-certificated at the Italian Ministry for Health.

It complies with the requirements of UNI EN ISO 10993 (parte 1, 5, 10): not irritant for skin, does not show any contact dermatitis allergic potential, non-toxic.

TECHNICAL CHARACTERISTICS

Fabric: production in Europe (UE)

Sizes: 98 x 180 mm

Weight: 3,5 g

1st liquid splash resistant layer: Hexagonal structure polyethylene net

2nd coloured outer layer: Cellulose and polyester, with oriented fibres

3rd filtering layer: Melt blown polypropylene fibrils

4th anti-macerating inner layer (in contact with the face): Polypropylene - Polyethylene

Edges: Non allergic white polypropylene

Nose clip: Metallic wire, diameter 0,53 mm, covered by white polypropylene, nuisance-free, adaptable, PVC-free

Nose clip length: 150 mm

Ear-loops: Tubular, made of polyester and elastomer, non-allergic and nuisance-free

Filtering efficiency (B.F.E.) (UNI EN 14683:2019): $\geq 99\%$

Resistance to breath (UNI EN 14683:2019): $< 30 \text{ Pa/cm}^2$

Bioburden (UNI EN 14683:2019): $\leq 30 \text{ UFC/g}$

Liquid splash resistance (UNI EN 14683:2019): Resistant at the pressure of 16.0 kPa

High resistance to microbes

Odourless totally
Extremely soft non-wovens
Fit for sensible skin
Disposable
Non sterile
Not irritant for skin

Does not contain:

- Glass fibres
- Latex
- PVC
- Phthalates
- Methylchloroisothiazolinone & Methylisothiazolinone
- Dimethyl fumarate
- Formaldehyde
- Allergens
- Heavy metals (antimony, arsenic, cadmium, mercury, lead)

USES

The Surgical Mask prevents environmental contamination by bacteria and secretions produced by the user and protects it from splashes of liquids.

It is intended for use:

- by medical staff
- by anyone who needs to reduce the risk of infection spread, in epidemic or pandemic situations
- In the industrial sector, for all environments where environmental pollution must be contained.

How to wear the mask?

Wash your hands before applying the mask.

Open the folds of the mask.

Place on the face: the opaque, hypoallergenic white side must be facing inwards in contact with the face with the moldable bar at the top.

Place the rubber bands behind the ears to fix the mask.

Tighten the moldable bar around the nose to minimize air spills.

In case of prolonged use, replace the mask every 2/3 hours to maintain the filtering efficiency.

Do not reuse after use.

Dispose of as normal waste.

STORAGE CONDITIONS

Keep away from direct solar rays and store in a fresh and dry place, in odourless area and away from heat sources.

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